The EZ-Change® Keeper & Cap Inserts For The AccessPost™ Overdenture System

ENGLISH: EDS has developed the EZ-Change® AccessPost™ Overdenture Attachment to facilitate easy replacement after wear. It consists of a metal keeper that is permanently cemented into the underside of the denture. The metal keeper has a space to receive a nylon cap insert that is threaded into place. When this nylon cap insert wears out, it takes just a few seconds to unscrew the worn out nylon cap insert from the metal keeper and replace it with a new one.

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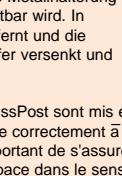
PLACEMENT TECHNIQUE

1.

ENGLISH: The AccessPost™ Overdenture posts are placed in the usual way. When the post fits properly within the post space, do not yet cement it in. It is important to make sure that when placed, that there is enough vertical space for the metal keeper and the nylon cap insert to be seated fully within the denture. To be sure, when the metal keeper with the nylon cap insert is first placed onto the ball attachment, prior to incorporation within the body of the denture, place marking spray onto the coronal surface of the metal keeper and place the denture over it. Remove the denture to view the area of impingement. Relieve the denture until there is no impingement and the denture is sitting passively on the ridge. In some cases it is not possible to relieve the denture from impingement without perforating the denture in an aesthetically crucial area. If this is the case, remove the uncemented post from the root and deepen the root-faced preparation. This allows for deeper seating of the post and more vertical clearance alleviating the possibility of an unaesthetic perforation of the denture.

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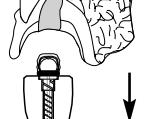
2.

ENGLISH: Once adequate vertical clearance is established, cement the post with EDS' composite cement, Flexi-Flow® with titanium (Catalog # 850-00) or Flexi-Flow Natural® (Catalog # 860-00).

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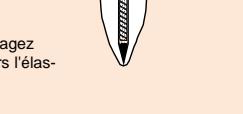
3.

ENGLISH: Place the metal keeper with the nylon cap insert onto the ball, making sure that the rubber band is in place.

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4.

ENGLISH: Cold cure the assembled metal keeper (with the nylon cap insert) into the underside of the denture.

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5.

ENGLISH: Remove the denture and relieve the excess acrylic. You may now remove the rubber band from the post's head.

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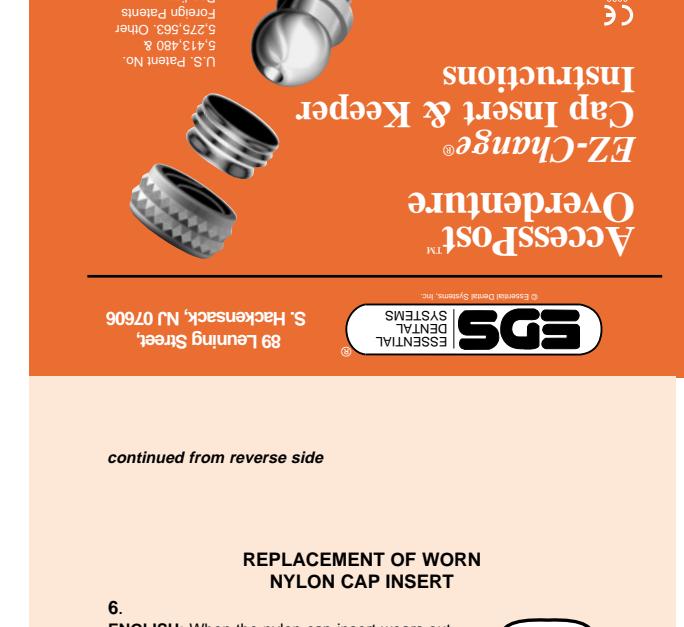
continued on reverse side . . .


continued from reverse side

REPLACEMENT OF WORN NYLON CAP INSERT

6.

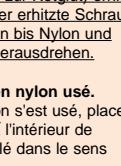ENGLISH: When the nylon cap insert wears out, take the two pronged wrench and place it into the two holes inside the nylon cap insert. Rotate the wrench counterclockwise for removal.

IMPORTANT: If the two holes are worn out from internal wear, heat the metal pronged wrench until the prongs are red hot and insert it anywhere into the nylon cap insert. The heated wrench will melt into the nylon creating its own holes. Allow a few seconds for the nylon to cool and solidify around the wrench. Then rotate the wrench counterclockwise until the nylon cap insert is removed.

Upon first removal and final seat, a tell-tale tactile click will be evident for the dentist to know when placement is complete.

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7.

ENGLISH: Place the two pronged wrench into a new nylon cap insert and rotate it into metal keeper until it clicks into place. Upon first removal and final seat, a tell-tale tactile click will be evident for the dentist to know when placement is complete.

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8.

ENGLISH: Place the denture onto the AccessPost Overdenture post(s) in the mouth. The denture should have the same fit as before. The replacement procedure of the nylon cap should have taken less than one minute.

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AccessPost Overdenture Kits and Their Contents

To order The AccessPost Overdenture System and accessories contact your authorized EDS dealer or call EDS.

AccessPost Overdenture Introductory Kits:

<