# Flexi-Flange® Fiber

Instruction Booklet for Flexi-Flange  
Fiber Posts

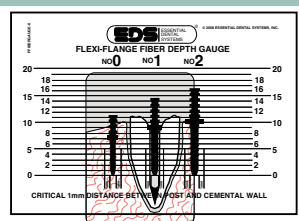
**NOTE:** If you are using Flexi-Flange® Fiber for the first time, you need the wrench, and depth gauge which are included in the Introductory Kit (Cat. No. 2410-00 or 2410-01), or may be purchased separately.

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**1. ENGLISH:** Use a depth gauge to determine optimum depth of the post. The second tier and flange of the post must be fully seated to achieve maximum stability, retention and resistance to fracture.

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Peso	Gates Glidden	EDS Gates Glidden	Primary Reamer Vorbohrer
1	2	0(YEL)	0(YEL)
3	4	1(RED)	1(RED)
4	5	2(BLU)	2(BLU)

**2. ENGLISH:** Use a Peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal (EDS® Gates Glidden Drills are recommended). Then use the primary reamer.

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**3. ENGLISH:** Use the primary reamer to prepare the full length of the post hole (Cut Wet).

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**4. ENGLISH:** Use a countersink drill/root facer to create the flange and second tier preparations. (The flange must be fully seated. If not, you reduce the post's stability and retention, and increase the chance of fracture under function.)

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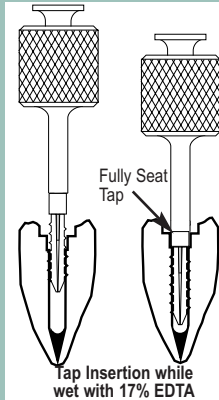
**5. ENGLISH:** Note: It is recommended that you flush the canal with 17% EDTA for 1 minute. (EDS recommends the use of its EDTA Plus, Cat. No. 770-16). This will remove debris and open the dentinal tubules, thereby increasing post retention. Tap while the canal is wet with 17% EDTA.

After choosing the correct size post with the depth gauge, the correct size tap that corresponds with that post is inserted. It is important to note that the tap is designed to be seated with light pressure. During the tap-seating, if moderate resistance is felt, the tap may be backed off 1/4 to 1/2 turn and then advanced again. Advancing while backing off 1/4 turn when moderate resistance occurs is repeated until the tap is fully inserted and the thread is created inside the root canal for the post. This procedure will remove debris from the thread line and facilitate insertion. NOTE: Please use appropriate autoclave sterilization methods such as heat or steam prior to each use. Additionally, periodic ultrasonic cleaning to remove debris from the split-shank. \*Important Note: If the primary reamer did not reach its full depth the tap will not seat fully. Tap the root to the length of the primary reamer and shorten the post apically so that the flange of the post is fully seated.

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**6. ENGLISH:** The correct size post is now ready to be trial seated. Using the Flexi-Flange Fiber wrench thread post with light pressure until it is fully seated. The post is now unthreaded out of the root. At this point, alterations to the post may be made.

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**7. ENGLISH:** Post adjustment: cut apical end, if necessary, to ensure sealing of the second tier. Note: apical adjustment can be done after trial insertion.

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**8. ENGLISH:** Dry the canal. Place composite resin cement in the canal - **No Bonding Agent is Required.** Place post on wrench and thread until fully seated. Remove the wrench and follow with core build-up and preparation. **Note fully seated flange (A).** (Flexi-Flow Cem® from EDS is recommended). Follow with core formation and preparation (Ti-Core® composite Core build-up material from EDS is recommended).

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For additional product information and latest instructions, please visit [www.edsdental.com](http://www.edsdental.com)

**Notes:** Keep posts out of sunlight. Prolonged direct exposure to sunlight will cause the Flexi-Post Fiber and Flexi-Flange Fiber to turn brown. In all Flexi-Post Fiber restorations, a minimum of a 1.5mm ferrule around the tooth is needed for the final restoration.

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WARNING: This product can expose you to the chemical Bisphenol A, which is known to the State of California to cause reproductive harm. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

Authorized European Representative  
European Healthcare & Devices Ltd.  
Stratton House  
Bishopstow Road  
Cork  
T12 Y9TK, Ireland  
Email: [TOMUK@aol.com](mailto:TOMUK@aol.com)

FOR MORE INFORMATION ON THESE AND ALL OUR INNOVATIVE PRODUCTS, OR TO ORDER CALL YOUR DENTAL DEALER

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

1-800-22-FLEXI

Tel: (201) 487-9090

Fax: (201) 487-5120

[www.edsdental.com](http://www.edsdental.com)

MADE IN USA

U.S. Patent No. 5,669,772 and U.S. and other foreign patents pending.

FFIBRE-3