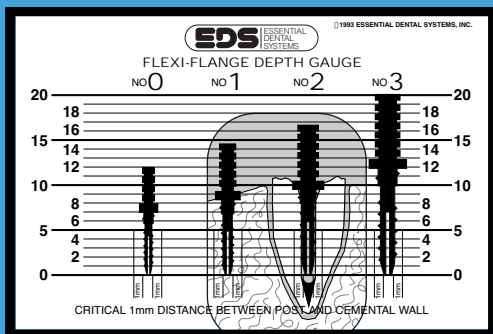
**NOTE:** If you are using Flexi-Flange® for the first time, you need the wrench, depth gauge, and extender which are included in the Introductory Kit (Cat. No. 410-00 or 415-00), or may be purchased separately.

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**1. ENGLISH:** Use a depth gauge to determine optimum depth of the post. Flange of the post must be fully seated to achieve maximum stability, retention and resistance to fracture.

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**2. ENGLISH:** Use a peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal. (EDS' Gates Glidden Drills are recommended.) Then use primary reamer.

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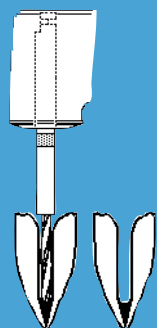
**3. ENGLISH:** Use the primary reamer to prepare the full length of the post hole (Cut Wet).

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
Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.

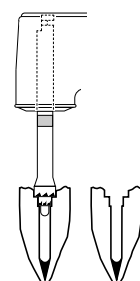ESSENTIAL DENTAL SYSTEMS


**4. ENGLISH:** Use a countersink drill/root facer to create the flange and second tier preparations. (The flange must be fully seated. If not, you reduce the post's stability and retention, and increase the chance of fracture under function.)

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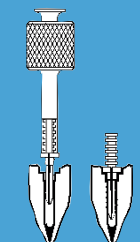
**5. ENGLISH:** Trial insert the post to determine fit.

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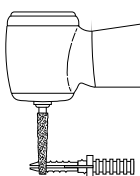
**6. ENGLISH:** Post adjustment: cut apical end, if necessary, to ensure seating of the flange. (Cut both legs at the same time.) Note: apical adjustment is always done after trial insertion.

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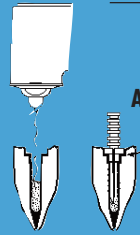**7. ENGLISH:** Post cementation. **Note fully seated flange, (A).** (Flexi-Flow® Cat. No. 850-00 or Flexi-Flow® Natural Cat. No. 860-00 cement from EDS is recommended.) Follow with core formation and preparation (Ti-Core® Cat. No. 800-00 or Ti-Core® Natural Cat. No. 810-00 composite Core build-up material from EDS is recommended).

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1-800-22-FLEXI  
Tel: (201) 487-9090

ESSENTIAL DENTAL SYSTEMS  
© Essential Dental Systems, Inc.

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.  
MADE IN USA

FFSSREI-9