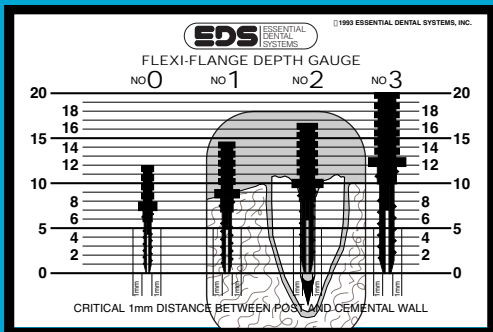
NOTE: If you are using Flexi-Flange® for the first time, you need the wrench, depth gauge, and extender which are included in the Introductory Kit (Cat. No. 410-00 or 415-00), or may be purchased separately.

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1. *Not shown actual size*

ENGLISH: Use a depth gauge to determine optimum depth of the post. Flange of the post **must be fully seated** to achieve maximum stability, retention and resistance to fracture.

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2. **ENGLISH:** Use a peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal. (EDS' Gates Glidden Drills are recommended.) Then use primary reamer.

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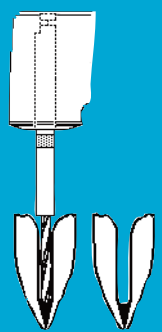
3. **ENGLISH:** Use the primary reamer to prepare the full length of the post hole (Cut Wet).

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MADE IN USA

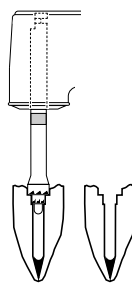
Flexi-Flange™
Genuine Split-Shank System
Refill Instruction Sheet

TITANIUM POSTS
TITAN WURZELSTIFT
PIVOTS EN TITANE
PERNI IN TITANIO
ESPIGA DE TITANIO

Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.

EDS ESSENTIAL DENTAL SYSTEMS


4. ENGLISH: Use a countersink drill/root facer to create the flange and second tier preparations. (The flange must be fully seated. If not, you reduce the post's stability and retention, and increase the chance of fracture under function.)

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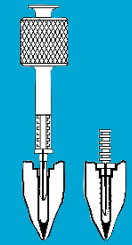
5. ENGLISH: Trial insert the post to determine fit.

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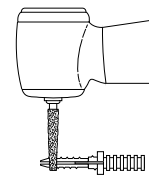6. ENGLISH: Post adjustment: cut apical end, if necessary, to ensure seating of the flange. (Cut both legs at the same time.) Note: apical adjustment is always done after trial insertion.

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7. ENGLISH: Post cementation. **Note fully seated flange, (A).** (Flexi-Flow® Cat. No. 850-00 or Flexi-Flow® Natural Cat. No. 860-00 cement from EDS is recommended.) Follow with core formation and preparation (Ti-Core® Cat. No. 800-00 or Ti-Core® Natural Cat. No. 810-00 composite Core build-up material from EDS is recommended).

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1-800-22-FLEXI
Tel: (201) 487-9090

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.
MADE IN USA

FR19E-7

Important Sterilization Procedures: **Endodontic Instruments, Posts, Drills and** **Wrenches are non-sterile.**

Prior to use, bag materials in an autoclave safe pouch and sterilize with a gravity displacement autoclave for 15 minutes at 132°C with a minimum drying time of 30 minutes. When reprocessing instruments, drills, wrenches, or taps, remove debris and dry before sterilization. Remove debris/soil using the EDS recommended cleaning protocol. Disinfect using a thermal disinfectant unit (washer-disinfectant) at 90°C for 1 minute.

For complete instructions see website.

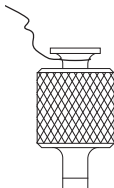
89 Leuning Street, S. Hackensack, NJ 07606
Tel: 1-800-22-FLEXI, 201-487-9090 • Fax: 201-487-5120
Web: www.edsdental.com E-Mail: info@edsdental.com

Non-Sterile

(OVER)

Post Wrench Procedures:

In addition to the placement of a rubber dam, attach/tie a piece of dental floss (24 inch minimum) to the groove on the top of the wrench handle to allow for retrieval of the wrench.