Flexi-Flow Cem® & Flexi-Flow Natural® Cement reinforced with titanium and lanthanide provide the compressive strength of dentin and proven long term fluoride release.

TO PREVENT PROBLEMS WITH MIXING:

- Always keep syringes tightly capped when not in use.
- Always dispose of plastic spatulas after using to prevent contamination.
- Always store at room temperature.
- Never return any gel to syringes after dispensing.

STEPS FOR MIXING

1. Dispense catalyst paste from the syringe onto the mixing pad.
2. Dispense an equal amount of base paste onto the same mixing pad.
3. With the wedge shaped end of the plastic spatula, mix the catalyst paste and the base past together for about 30 seconds to achieve thorough blending. (Flexi-Flow gives you 3 minutes working time before it sets.)
4. To make Flexi-Flow set faster, add more base. To make Flexi-Flow set slower, add more catalyst.

MIX, PLACE, SET, HARDEN — ALL IN JUST FIVE MINUTES

After mixing Flexi-Flow Cem , place it onto the post and into the post-hole using a Lentulo spiral drill, or any instrument you feel appropriate. Insert post, let set undisturbed for the next 4 minutes. The post is now ready for core build-up.

Note:

- a. To make use of the maximum tensile strength of Flexi-Flow when used for cementing posts, the canal should be etched or grooved (use Essential Dental Systems' Grooving Drills, Cat. No. 593-0 or Cat. No. 593-123).
- b. Flexi-Flow should not be used with any bonding agent in the canal. Bonding agents within the canal greatly accelerate the setting time of Flexi-Flow / Flexi-Flow Natural.

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