Flexi-Flow Natural

Recommended for prefabricated post systems.

REINFORCED COMPOSITE CEMENT

Flexi-Flow Natural is a composite cement with patented reinforcing agents, that has a compressive strength and aesthetics similar to dentin. It was developed to enhance the retention of all prefabricated posts. The unique ingredients in the base and catalyst gel, mix together to create a composite with the low viscosity you want, and the compressive strength and permanence that post placement requires.

UNIQUE FEATURES OF FLEXI-FLOW NATURAL

- Compressive strength of 46,511 p.s.i. (comparable in strength to natural dentin).
- Natural tooth color formulation to match Shade Vita™ A3
- Looks like dentin so it can be used under porcelain restorations.
- Low viscosity gel consistency.
- Chemically bonds to composite core build-up, eliminating time consuming removal of excess cement.
- Reduces hydrostatic pressure, and coats the post-hole and post effectively.
- Takes just FIVE MINUTES to mix, place, set and harden.

TO PREVENT PROBLEMS WITH MIXING

- ALWAYS keep syringes tightly capped when not in use.
- ALWAYS dispose of plastic spatulas after using to prevent contamination.
- ALWAYS store at room temperature.
- NEVER return any gel to syringes after dispensing.

STEPS FOR MIXING

- Dispense catalyst paste from the syringe onto the mixing pad.
- Dispense an equal amount of base paste onto the same mixing pad.
- With the wedge shaped end of the plastic spatula, mix the catalyst paste and the base paste together for about 30 seconds to achieve thorough blending. (Flexi-Flow Natural gives you 3 minutes working time before it sets.)
- To make Flexi-Flow set faster, add more base.**
To make Flexi-Flow Natural set slower, add more catalyst.

MIX, PLACE, SET, HARDEN –ALL IN JUST FIVE MINUTES

After mixing Flexi-Flow Natural for 30 seconds, place it onto the post and into the post-hole using a Lentulo spiral drill, or any instrument you feel appropriate. Insert post, let set undisturbed for the next 4 minutes.

The post is now ready for core build-up.

- NOTES:**
- To make use of the maximum tensile strength of Flexi-Flow Natural when used for cementing posts, the canal should either be etched or grooved (Use EDS’ Grooving Drills, Cat. No. 593-0 or Cat. No. 593-123.)**
 - Flexi-Flow should not be used with any bonding agent in the canal. Bonding agents within the canal greatly accelerate the setting time of Flexi-Flow Natural Cement.**

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- Base Gel: 9gm
- Plastic Spatulas
- Catalyst Gel: 9gm
- Instructions
- Mixing Pad

860-00

REF

Cat. No.

0086

CE

ESSENTIAL DENTAL SYSTEMS

EC REP

Authorized European Representative:
Research Affairs, OMR,
European Device Solutions
33, Stapleford Road,
Middlebrough, Cleveland
Leeds, England
Contact E-Mail: TOMURK@aol.com

Manufactured for:
Essential Dental Systems, Inc.
89 Leuning Street, S.
Hackensack, NJ 07606
www.edsdental.com

Flexi-Flow[®]
NATURAL

Composite Cement • Shade Vita™ A3 for the look of natural dentin • Radiopaque
Cemento Composito • Shade Vita™ A3 para una dentina natural • Radiopaco
Ciment Composite • Shade Vita™ A3 pour l' aspect de la dentine naturelle • Radiopaque
Composit-Zahnzement • Farbtonung Vita™ A3 sieht sie natürliches Dentin aus • Radiopak
Cemento Composito • Shade Vita™ A3 per l' aspetto della dentina naturale • Radiopaco

ESSENTIAL DENTAL SYSTEMS

EDS

ESSENTIAL DENTAL SYSTEMS

DL 10/12 © Essential Dental Systems, Inc.
89 Leuning Street, S. Hackensack, NJ 07606 USA

1-800-22-FLEXI
(35394)

Tel: (201) 487-9090 Fax: (201) 487-5120

www.edsdental.com

U.S. Patent Nos. 5,034,433 & 5,204,398.
European Patent No. EP 0 631 598 B1. Other patents pending.