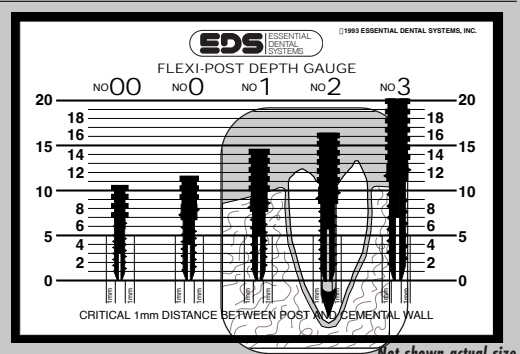
**NOTE:** If you are using Flexi-Post® for the first time, you need the wrenches, depth gauge and extender which are included in the Introductory Kit (Cat. No. 110-00 or 110-01), or may be purchased separately.

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**1. ENGLISH:** Use a depth gauge to determine optimum depth of the post. The second tier of the post must be fully seated to achieve maximum stability, retention and resistance to fracture.

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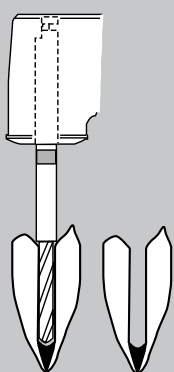
**2. ENGLISH:** Use a Peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal (EDS® Gates Glidden Drills are recommended). Then use the primary reamer.

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**3. ENGLISH:** Use the primary reamer to prepare the full length of the post hole. (Cut Wet).

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Caution: Federal law restricts this product to use by or on the order of a licensed dentist.


Refill Instruction Sheet

Genuine Split-Shank System™

**Flexi-Post™**

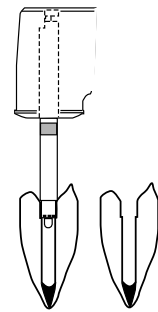TITANIUM POSTS  
PIVOTS EN TITANE  
PERNI IN TITANIO  
ESPIGAS DE TITANIO

ESSENTIAL DENTAL SYSTEMS


**4. ENGLISH:** Use the secondary drill to prepare the seat for the second tier. (The second tier must be fully seated. If not, you reduce the Flexi-Post's retention and increase the chance of fracture under function.)

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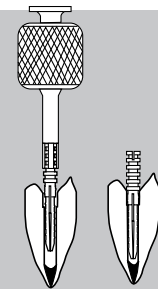
**5. ENGLISH:** Trial insert the post to determine fit.

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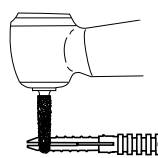**6. ENGLISH:** Post adjustment: cut apical end, if necessary, to ensure seating of the second tier. (Cut both legs at the same time.) Note: apical adjustment is always done after trial insertion.

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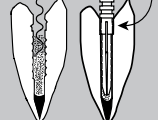
**7. ENGLISH:** Post cementation. **Note fully seated second tier (A).** (Flexi-Flow™ Cat. No. 850-00 or Flexi-Flow™ Natural Cat. No. 860-00 cement from EDS is recommended.) Follow with core formation and preparation (Ti-Core™ Cat. No. 800-00 or Ti-Core™ Natural Cat. No. 810-00 composite Core build-up material from EDS is recommended).

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1-800-22-FLEXI  
Tel: (201) 487-9090

ESSENTIAL DENTAL SYSTEMS

© Essential Dental Systems, Inc.  
89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.  
MADE IN USA

FPRIN-6