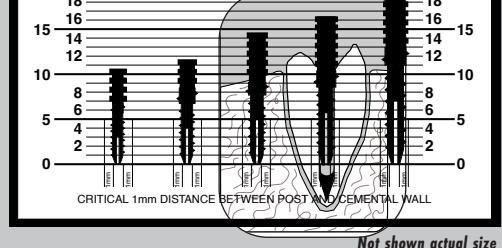
NOTE: If you are using Flexi-Post® for the first time, you need the wrenches, depth gauge and extender which are included in the Introductory Kit (Cat. No. 110-00 or 110-01), or may be purchased separately.

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1.
ENGLISH: Use a depth gauge to determine optimum depth of the post. The second tier of the post must be fully seated to achieve maximum stability, retention and resistance to fracture.

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2.
ENGLISH: Use a Peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal (EDS' Gates Glidden Drills are recommended). Then use the primary reamer.

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3.
ENGLISH: Use the primary reamer to prepare the full length of the post hole (Cut Wet).

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FLEXI-POST®

Refill Instruction Sheet

Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.

ESPIGA DE ACERO INOXIDABLE
PEÑA IN ACCIAIO INOSSIDABILE
PIVOT EN ACIER INOSSIDABLE
PROFIL AUS ROSTFREiem STAHL
STAINLESS STEEL POSTS

SGS DENTAL SYSTEMS

4. ENGLISH: Use the secondary drill to prepare the seat for the second tier.
(The second tier must be fully seated. If not, you reduce the Flexi-Post's retention and increase the chance of fracture under function.)

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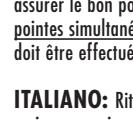
5. ENGLISH: Trial insert the post to determine fit.

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6. ENGLISH: Post adjustment: cut apical end, if necessary, to ensure seating of the second tier.
(Cut both legs at the same time). Note: apical adjustment is always done after trial insertion.

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7. ENGLISH: Post cementation. Note fully seated second tier (A). (Flexi-Flow Cem® from EDS is recommended). Follow with core formation and preparation (Ti-Core® composite Core build-up material from EDS is recommended).

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© 1993 Essential Dental Systems, Inc. H-1
89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.