1.
ENGLISH: Depth gauge to determine optimum depth of post. Flange of the post must be fully seated to achieve maximum stability, retention and resistance to fracture.

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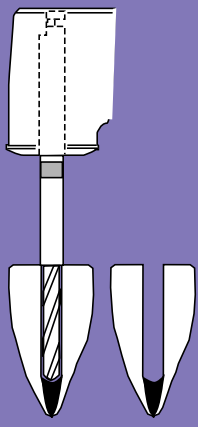
2.
ENGLISH: Peeso or Gates Glidden drill to remove gutta percha, and for preliminary sizing of canal. (EDS' Gates Glidden Drills are recommended). Then use primary reamer.

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
3.
ENGLISH: Primary reamer to prepare full length of post hole. (Cut wet).

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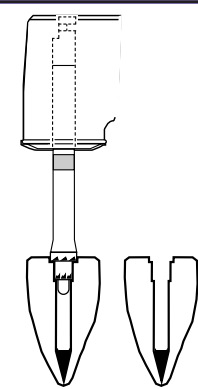
4.
ENGLISH: Countersink drill/root facer to create the flange and second tier preparations. (The flange and second tier must be fully seated. If not, you reduce the post's stability and retention and increase the chance of fracture under function).

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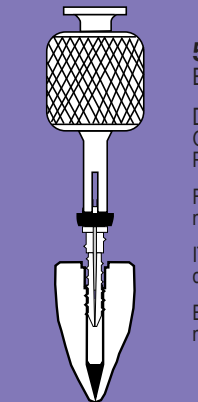
5.
ENGLISH: Trial insertion to determine fit.

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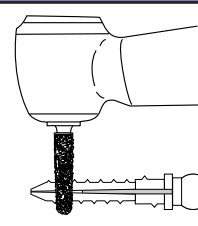6.
ENGLISH: Post adjustment: cut apical end, if necessary, to ensure seating of flange. (Cut both legs at the same time). Note: apical adjustment is always done after trial insertion.

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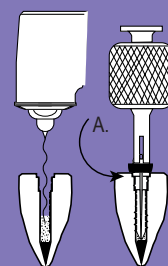
7.
ENGLISH: Post cementation. Note fully seated flange & second tier (A).

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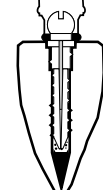
8.
ENGLISH: Place nylon female cap on ball of post. Make sure the rubber band is covering the height of contour of the head. If not, there is a risk that the cold cured acrylic could lock in under the head, making the removal of the denture difficult.

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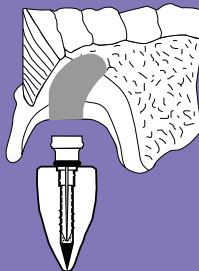
9.
ENGLISH: Relieve denture to make room for cap.

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
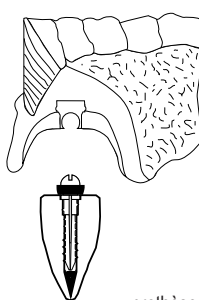
10.
ENGLISH: Place pink cold cure acrylic in relieved denture. Seat denture, let set, and pick up attachment cap.

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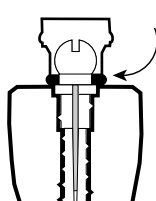
NOTE: The cap should have a little clearance from the root when seated on the ball of the post so that it will rotate. If not, simply relieve a small amount of nylon on lip of cap; and if necessary, relieve some peripheral tooth structure to create space.

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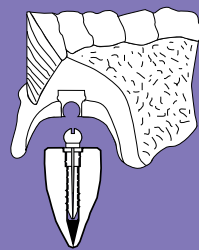11.
ENGLISH: Remove rubber band and trim flash. Overdenture is now ready for use.

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ALSO AVAILABLE: EZ-CHANGE® (Cat. #250-00) A patented keeper and cap insert system that allows for "quick & easy" nylon cap insert replacement. When worn, this cap insert may be easily replaced in seconds.

1-800-22-FLEXI

Tel: (201) 487-9090

© Essential Dental Systems, Inc.

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

MADE IN USA

Important Sterilization Procedures: **Endodontic Instruments, Posts, Drills and** **Wrenches are non-sterile.**

Prior to use, bag materials in an autoclave safe pouch and sterilize with a gravity displacement autoclave for 15 minutes at 132°C with a minimum drying time of 30 minutes. When reprocessing instruments, drills, wrenches, or taps, remove debris and dry before sterilization. Remove debris/soil using the EDS recommended cleaning protocol. Disinfect using a thermal disinfectant unit (washer-disinfectant) at 90°C for 1 minute.

For complete instructions see website.

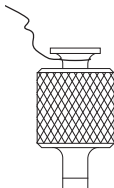
89 Leuning Street, S. Hackensack, NJ 07606
Tel: 1-800-22-FLEXI, 201-487-9090 • Fax: 201-487-5120
Web: www.edsdental.com E-Mail: info@edsdental.com

Non-Sterile

(OVER)

Post Wrench Procedures:

In addition to the placement of a rubber dam, attach/tie a piece of dental floss (24 inch minimum) to the groove on the top of the wrench handle to allow for retrieval of the wrench.