
INDIRECT/COPING TECHNIQUE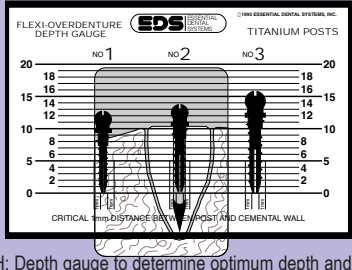

1. ENGLISH: Depth gauge to determine optimum depth and width of post.

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2. ENGLISH: Peeso or Gates Glidden reamer to remove gutta-percha, and for preliminary sizing of the canal (EDS' Gates Glidden Drills are recommended). Then use primary reamer.

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3. ENGLISH: Primary reamer to prepare full length of post hole. (Cut Wet).

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4. ENGLISH: Secondary drill to prepare seat for second tier.

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5. ENGLISH: Trial insertion to determine fit. Flange of post must be fully seated on the coronal tooth structure.

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6. ENGLISH: Post adjustment: Remove post. Cut apical end if necessary, to ensure seating of second tier and coronal contact of flange. (Cut both legs at the same time). Note: apical adjustment is always done after trial insertion.

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7. ENGLISH: Reset the post, making sure there is a 0.5mm space between the flange and the coronal tooth structure for the coping to be placed. To achieve this space, seat the post completely, and back off 1/2 turn.

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8. ENGLISH: Take an impression of the trial seated post (the post is not cemented in at this point).

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9. ENGLISH: Separate the impression from the post.

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10. ENGLISH: Remove the post from the canal and temporarily seal the canal.

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11. ENGLISH: Send the impression to the laboratory along with the appropriate size brass transfer stud. (#of rings = size of post).

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12. ENGLISH: The laboratory lubricates and places the appropriate transfer stud in the impression, and pours it up in stone or plaster.

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13. ENGLISH: The laboratory then waxes up and casts the coping.

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14. ENGLISH: The dentist cements the coping (a). While the cement is still wet the Flexi-Overdenture post is cemented in place (b). (Ti-Core Flow+ composite cement from EDS is recommended). The insertion stops when the flange is fully seated within the coping.

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Flexi-Overdenture® Refill Instruction Sheet For Indirect Technique and Direct

TITANIUM POSTS
PIVOTS EN TITANE
PERNI IN TITANIO
ESPIGA DE TITANIO

15. ENGLISH: The dentist is now ready to insert the female cap into the denture. Place nylon female cap on ball of post. Make sure the rubber band is covering the height of contour of the head. If not, there is a risk that the cold cured acrylic could lock in under the head, making removal of the denture difficult.

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16. ENGLISH: Relieve denture to make room for cap.

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17. ENGLISH: Place pink cold cure acrylic in relieved denture. Seat denture, let set, and pick up attachment cap.

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NOTE: The cap should have a little clearance from the coping or root when seated on the ball of the post so that it will rotate. If not, simply relieve a small amount of nylon on lip of cap; and if necessary, relieve some coping or peripheral tooth structure to create space.

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18. ENGLISH: Remove rubber band and trim flash. Overdenture is now ready for use.

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1 - 3. DIRECT/NON-COPING TECHNIQUE

ENGLISH: Follow steps 1 - 3 of "Indirect/coping technique" (see reverse side of sheet).

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ENGLISH: Use OVD countersink drill/root facer to create the flange and second tier preparations. (The flange and second tier must be fully seated. If not, you reduce the post's stability and retention and increase the chance of fracture under function.)

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5. ENGLISH: Trial insertion to determine fit.

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6. ENGLISH: Post Adjustment: Remove post. Cut apical end, if necessary, to ensure seating of flange and second tier. (Cut both legs at the same time). Note: apical adjustment is always done after trial insertion.

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7. ENGLISH: Post cementation. Note: fully seated flange & second tier (A). (Ti-Core Flow+ composite cement from EDS is recommended).

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8. ENGLISH: Follow instructions 15 to 18 of "Indirect/coping technique" for cap incorporation (see reverse side of sheet).

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
ALSO AVAILABLE: EZ-CHANGE (Cat. #250-00) A patented keeper and cap insert system that allows for quick & easy nylon cap insert replacement. When worn, this cap insert may be easily replaced in seconds.

1-800-22-FLEXI
Tel: (201) 487-9090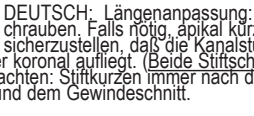

Copyright © 1998 Essential Dental Systems, Inc. All rights reserved. Printed in the USA.

Important Sterilization Procedures: **Endodontic Instruments, Posts, Drills and** **Wrenches are non-sterile.**

Prior to use, bag materials in an autoclave safe pouch and sterilize with a gravity displacement autoclave for 15 minutes at 132°C with a minimum drying time of 30 minutes. When reprocessing instruments, drills, wrenches, or taps, remove debris and dry before sterilization. Remove debris/soil using the EDS recommended cleaning protocol. Disinfect using a thermal disinfectant unit (washer-disinfectant) at 90°C for 1 minute.

For complete instructions see website.


89 Leuning Street, S. Hackensack, NJ 07606
Tel: 1-800-22-FLEXI, 201-487-9090 • Fax: 201-487-5120
Web: www.edsdental.com E-Mail: info@edsdental.com

Non-Sterile

(OVER)

Post Wrench Procedures: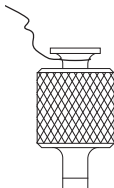

In addition to the placement of a rubber dam, attach/tie a piece of dental floss (24 inch minimum) to the groove on the top of the wrench handle to allow for retrieval of the wrench.