
INDIRECT/COPING TECHNIQUE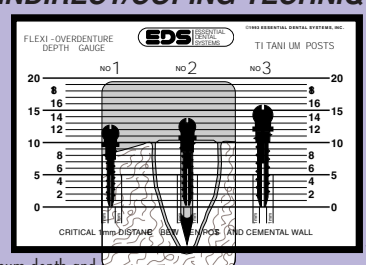

ENGLISH: Depth gauge to determine optimum depth and diameter of post.

1. ENGLISH: Use the depth gauge to determine optimum depth and diameter of post.

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ENGLISH: Primary reamer to prepare full length of post hole. (Cut Wet).

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ENGLISH: Secondary drill to prepare seat for second tier.

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ENGLISH: Trial insertion to determine fit. Flange of post must be fully seated on the coronal tooth structure.

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ENGLISH: Post adjustment: Remove post. Cut apical end if necessary, to ensure seating of second tier and coronal contact of flange. (Cut both legs at the same time). Note: apical adjustment is always done after trial insertion.

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ENGLISH: Reseat the post, making sure there is a 0.5mm space between the flange and the coronal tooth structure for the coping to be placed. To achieve this space, seat the post completely, and back off 1/2 turn.

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ENGLISH: Take an impression of the trial seated post (the post is not cemented in at this point).

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ENGLISH: Separate the impression from the post.

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ENGLISH: Remove the post from the canal and temporarily seal the canal.

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ENGLISH: Send the impression to the laboratory along with the appropriate size brass transfer stud. (#of rings = size of post).

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ENGLISH: The laboratory lubricates and places the appropriate transfer stud in the impression, and pours it in stone or plaster.

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ENGLISH: The laboratory then waxes up and casts the coping.

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ENGLISH: The dentist cements the coping (a). While the cement is still wet the Flexi-Overdenture post is cemented in place (b). (Ti-Core Flow+ composite cement from EDS is recommended).

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For Indirect Technique and Direct Technique

Refill Instruction Sheet

Flexi-Overdenture

TITANIUM POSTS

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PIVOTS EN TITANE

FRANZI IN TITANIO

ESPIGA DE TITANIO

ESSENTIAL DENTAL SYSTEMS

89 Leuning St., Hackensack, NJ USA

ENGLISH: The dentist is now ready to insert the female cap into the denture. Place nylon female cap on ball of post. Make sure the rubber band is covering the height of contour of the head. If not, there is a risk that the cold cured acrylic could lock in under the head, making removal of the denture difficult.

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ENGLISH: Relieve denture to make room for cap.

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ENGLISH: Place pink cold cure acrylic in relieved denture. Seat denture, let set, and pick up attachment cap.

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NOTE: The cap should have a little clearance from the coping or root when seated on the ball of the post so that it will rotate. If not, simply relieve a small amount of nylon on lip of cap; and if necessary, relieve some coping or peripheral tooth structure to create space.

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ENGLISH: Remove rubber band and trim flash. Overdenture is now ready for use.

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1 - 3. DIRECT/NON-COPING TECHNIQUE

ENGLISH: Follow steps 1 - 3 of "Indirect/coping technique" (see reverse side of sheet)

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ENGLISH: Use OVD countersink drill/root facer to create the flange and second tier preparations. (The flange and second tier must be fully seated. If not, you reduce the post's stability and retention and increase the chance of fracture under function.)

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ENGLISH: Trial insertion to determine fit.

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ENGLISH: Post Adjustment: Remove post. Cut apical end, if necessary, to ensure seating of the flange and second tier. (Cut both legs at the same time). Note: apical adjustment is always done after trial insertion.

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ENGLISH: Post cementation. Note fully seated flange & second tier (A). (Ti-Core Flow+ composite cement from EDS is recommended).

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ENGLISH: Follow instructions 15 to 18 of "Indirect/coping technique" for cap incorporation (see reverse side of sheet).

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ALSO AVAILABLE: EZ-CHANGE (Cat. #250-00) A patented keeper and cap insert system that allows for "quick & easy" nylon cap insert replaced in seconds.

© Essential Dental Systems, Inc.

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

MADE IN USA

1-800-22-FLEXI

Tel: (201) 487-9090

ESSENTIAL DENTAL SYSTEMS

Authorized European Representative
European Healthcare & Devices Ltd
Stratton House, Bishopstow Road
Cork
T12 Y9TC, Ireland
Email: TQMUK@aol.com

EC REP

1-800-22-FLEXI

Tel: (201) 487-9090

ESSENTIAL DENTAL SYSTEMS

© Essential Dental Systems, Inc.

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

MADE IN USA

1-800-22-FLEXI

Tel: (201) 487-9090

ESSENTIAL DENTAL SYSTEMS

Authorized European Representative
European Healthcare & Devices Ltd
Stratton House, Bishopstow Road
Cork
T12 Y9TC, Ireland
Email: TQMUK@aol.com

EC REP

1-800-22-FLEXI

Tel: (201) 487-9090

ESSENTIAL DENTAL SYSTEMS

© Essential Dental Systems, Inc.

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

MADE IN USA

1-800-22-FLEXI

Tel: (201) 487-9090