Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.

89 Leuning Street, S. Hackensack, NJ 07606
 Tel: 1-800-22-FLEXI, 201-487-9090 • Fax: 201-487-5120
 Web: www.edsdental.com E-Mail: info@edsdental.com

Patent No. 6,042,376 and other U.S. and foreign patents pending. Non-Sterile

PulpOut Bur Endodontic Access Kit Instructions

Research has shown that there is a critical depth of 7 mm from the cusp tip to the pulp chamber of teeth with furcations. The PulpOut bur allows you to take advantage of this depth and gain access without furcation perforation.

Instructions for normal & calcified pulp chambers

***Note: One or more CUSP TIPS of the tooth must be intact (either natural or crowned) in order for the PulpOut bur to attain the proper depth. If a tooth is either missing the cusps for measurement or the cusps are appreciably worn down from attrition or the tooth is prepared for a crown DO NOT use the PulpOut bur.**

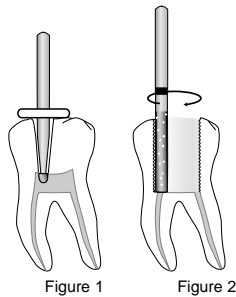
Figure 1

Figure 2

Important:

Make sure to cut wet using water spray.

1. Using water spray, in teeth with furcations and at least one intact cusp tip, drill the access opening into the pulp. If the tooth is restored with a porcelain restoration use a diamond first to go through the porcelain. The correct depth has been reached when the circular stop of the "PulpOut" drill is prevented from further apical movement by any of the remaining cusp tips. See figure 1.

2. Make a rough outline of the pulp chamber using the "PulpOut" bur.

3. Refine the axial walls of the access preparation using the non-end cutting diamond "shaper" bur. See figure 2. This is done by leaning the diamond shaper against the axial wall and going around the entire access opening outline. This smoothes the axial walls, lets more light in and **makes it easier to find the canals.**

4. You are now ready to look for the canals. (EDS recommends shaping the canals with the SafeSiders® instrumentation system).

Note: For up-to-date instructions on this, and all EDS products, please check our web site at www.edsdental.com.

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