

Cleaning and Lubrication

Important:

If lubrication instructions are not followed you may damage the handpiece and void warranty.

Use aerosol spray lubricant after each use and/or before autoclaving.

1. Push E-type spray nozzle attachment over the aerosol spray lubricant nozzle until it firmly seats (Fig 1). Shake the can 3-4 times to mix lubricant and propellant.
2. Insert the E-type spray nozzle in the rear of the handpiece and spray for approximately 2-3 seconds until the oil comes out of the handpiece head or nose (Figs 2-3).
3. Run the attachment for 30 seconds so that all excess fluid is expelled. This ensures all internal parts of the attachment are completely cleaned.

E-Type Spray Nozzle Aerosol Spray Lubricant Endo-Express Handpiece



Fig. 1

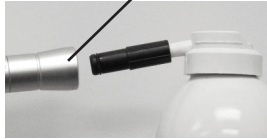


Fig. 2



Fig. 3



Cautions on Lubrication

- Be sure to hold the handpiece firmly to prevent the handpiece from slipping out by the spray pressure when lubricating.
- Supply lubricant until it comes out of the handpiece nose (for approx. 2 seconds).
- Keep the aerosol spray lubricant can upright.

Sterilization

EDS recommends heat sterilization by autoclave prior to first use and after each use to prevent cross-contamination between patients.

Autoclave Procedure:

1. Scrub dirt and debris from the handpiece, and wipe clean with alcohol-immersed cotton swab or cloth. Do not use a wire brush.
2. Lubricate with aerosol spray lubricant. Refer to the Lubrication Instructions.
3. Insert into an autoclave pouch. Seal the pouch.
4. Steam sterilize per Gravity procedures below:
 - Temperature: 270° F/132° C. • Cycle Time: 15 minutes • Maximum Dry Time: 20 minutes
5. Keep the handpiece in the autoclave pouch to keep it clean until you use it.



Cautions on Sterilization

- Do not exceed 275° F (29 psi)/135° C (2 bar).
- Always place the handpiece in the center or upper half of the chamber, as the local temperature at the chamber bottom could rise beyond the set value.
- Do not autoclave the handpiece in the same bag as other instruments or equipment.
- Do not immerse instrument in water or ultrasonic cleaner.
- Do not use any type of disinfectant, chemical or soap on instrument. Use of chlorine products, aldehydes, will damage attachments and void all warranties.
- Do not dry heat or heat transfer sterilize.



89 Leuning Street
S. Hackensack, NJ 07606
Phone: 1-800-22-FLEXI • 201-487-9090
Fax: 201-487-5120
www.edsdental.com

