# TI-CORE®

## NATURAL (Tooth Color Vita® A3)

### Core Material

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CATALOG NUMBER: **810-00**  
REF:

CONTENTS: • ONE 15 GRAM JAR OF BASE  
• ONE 15 GRAM JAR OF CATALYST  
• 50 DISPOSABLE SPATULAS  
• 100 MIXING PAD SHEETS  
• INSTRUCTION BOOKLET

STORE BETWEEN 60°-85°F / 15°-29°C

Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.

EC REP

Authorized European Representative  
European Healthcare & Devices Ltd.  
Stratton House  
Bishopstown Road  
T12 Y9TC, Ireland  
Email: TOMUK@aol.com

#### English:

Ti-Core Natural is a reinforced, hybrid filled, autocuring composite, that creates a unique combination of great strength and aesthetics. The addition of patented reinforcing agents increase the composite's compressive strength comparable to that of dentin. For the first time, you have a core material that provides strength and aesthetics comparable to dentin. The clinical significance of Ti-Core Natural means the elimination of ditching during core preparation, and long term success of the restoration under function.

#### Precautions:

- Contact with eyes and skin should be avoided. If accidental contact occurs rinse immediately with water and consult a physician if necessary.
- Do not use materials containing eugenol it will delay or inhibit the set.
- Water and oily air inhibit polymerization.
- There are no known systemic side effects to date, however, do not use in patients with known sensitivities to Bis-GMA, benzyl peroxide or composite resin materials.

Please Note: Mixing tips and mixing pad sheets are single use only. If necessary, the exterior of the syringe may be disinfected with alcohol or sodium hypochlorite. Do NOT autoclave.

To download Material Safety Data Sheet (MSDS) / SDS (Safety Data Sheets) Please Visit: [Http://edsdental.com/sds](http://edsdental.com/sds)

#### AREAS OF USE:

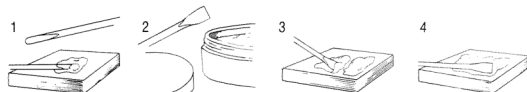
1. NONVITAL TEETH - Ti-Core Natural is recommended for use with all prefabricated posts and/or pin build-ups.
2. VITAL TEETH - Ti-Core Natural may be used with pins to build-up missing tooth structure. A calcium hydroxide or glass ionomer base may be used under the Ti-Core Natural composite.

#### BONDING TO ENAMEL & DENTIN:

- a. Ti-Core Natural can be bonded to enamel with the acid etch technique.
- b. Ti-Core Natural can be bonded to dentin using any of the available multi-step dentin bonding agents currently on the market.

#### MIXING TI-CORE NATURAL:

1. Using the rounded end of the disposable plastic spatula, remove from the jar of CATALYST PASTE an amount slightly less than the volume of the cavity to be filled. Place on mixing pad.
2. Using the flat, squared end of the disposable plastic spatula, remove for the jar of BASE PASTE an amount approximately equal to the Catalyst Paste placed on the mixing pad in step 1.
3. Place Base Paste next to Catalyst Paste on mixing pad. REPLACE CAPS ON JARS.
4. Using the flat, squared end of the disposable plastic spatula, mix the two pastes for about 20 seconds using 1 to 2 strokes per second. The objective is to make the mixture as homogenous as possible.
5. To make Ti-Core Natural set faster, **add more base**. To make Ti-Core Natural set slower, **add more catalyst**.

#### PLACEMENT:

1. To create an intimate adaptation between root, post and core material, place Ti-Core into a core form and seat over the root and post with firm pressure.
2. Alternatively, you can place a matrix band around the prepared tooth and post, syringe Ti-Core and apply light pressure for an intimate fit.

#### TOOTH PREPARATION:

After six minutes for the beginning of mixing the core build-up material, remove the core form or matrix band. Diamond or carbide burs may be used to prepare the tooth. Air and water cooling is recommended to cut the material more efficiently.

MIX TIME	PLACEMENT TIME	SETTING TIME (DO NOT DISTURB)	HARDENING TIME	REMOVE MATRIX, FINISH AND POLISH
20 SEC.	1 MIN. 40 SEC.	2 MIN.	2 MIN.	

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