



Endodontic Technique Tips to Remember



ALWAYS IRRIGATE WELL DURING INSTRUMENTATION

Irrigating not only helps to remove bacteria and debris from the canal, it helps lubricate and facilitate canal instrumentation.



WIPE THE INSTRUMENT DURING INSTRUMENTATION

By wiping the instrument you remove debris from the flutes and facilitate efficient and safe instrumentation.



USE A SHORT PECKING MOTION

By using a short 2 - 3 mm pecking motion, you move debris up which helps not impact debris apically.



MAKE SURE THE INSTRUMENT FITS LOOSELY IN THE CANAL

After you have achieved working length, work the lateral walls, instrumenting until it is loose and patent in the canal before moving to the next instrument in sequence.



USE THE CORRECT HANDPIECE

Only use Tango-Endo Instruments in the Tango-Endo Handpiece. Other handpieces may cause instrument separation.



DON'T OVERUSE THE INSTRUMENTS

Tango-Endo Instruments should not be used for more than 3 canals before replacing.