


QUICK GUIDE TO OPERATING THE AEU-17BEA (SERGIMOTOR) TULSA/DENTSPLY

CONTROL PANEL DIAGRAM

AEU-17BEA Manual

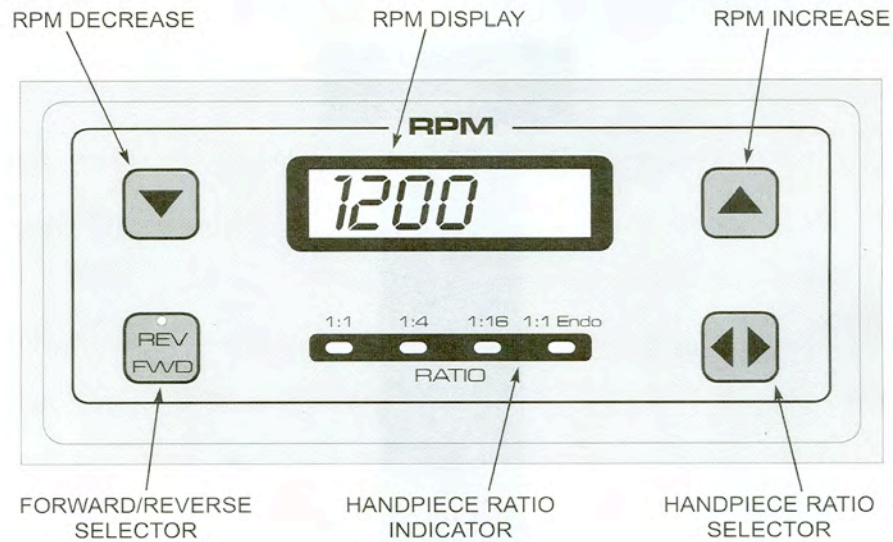

The AEU-17BEA **cannot** be preset. The motor must be set manually each time.

Setting the AEU-17BEA for Endo-Express.

1. Turn on the power switch.
2. Push the up arrow to **8000**.
3. The Forward/Reverse Selector should always be **Forward**.
4. Push the Left/Right arrows on the Handpiece Ratio Selector to either **1:1, 1:4, or 1:1 ENDO**.
5. This unit does **not** have an Auto Reverse Function.