


**QUICK GUIDE TO OPERATING THE
AEU-20T (ITR) TULSA/DENTSPLY**

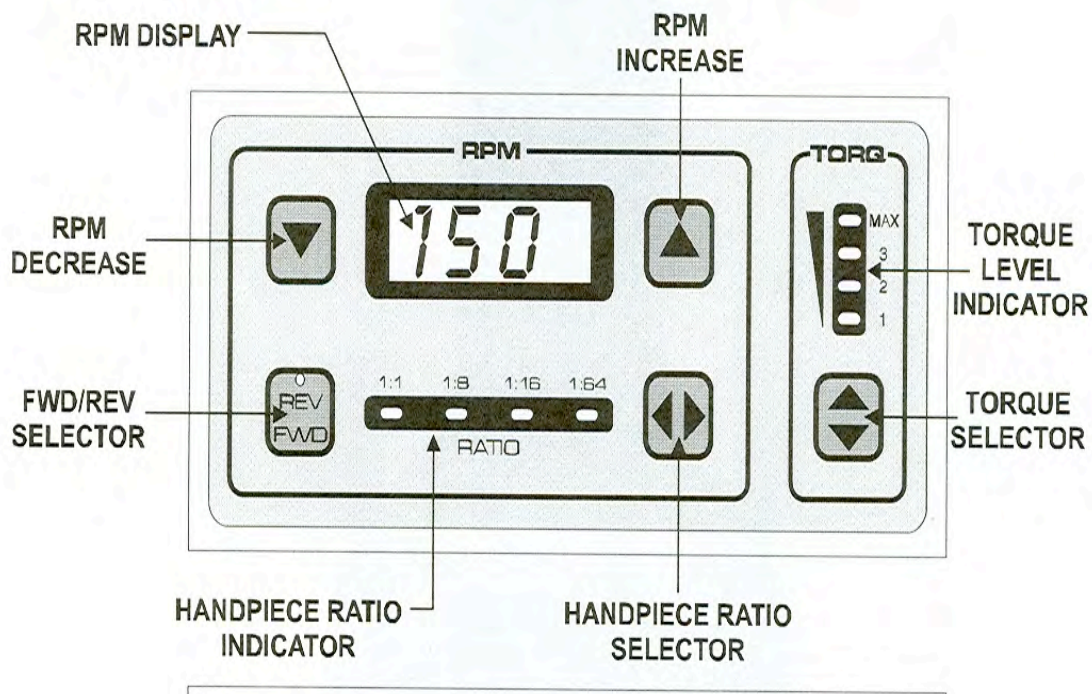

The AEU-20T (ITR) motor **cannot** be preset. The motor must be set manually each time.

Setting the AEU-20 ITR for Endo-Express.

1. Turn on the power switch.
2. Adjust the up/down arrows on the RPM display to **8000**.
3. The Forward/Reverse selector should always be **forward**.
4. Adjust the left/right arrows on the Handpiece Ratio Selector to **1:1**.
5. Adjust the up/down arrows on the Torque Selector to **MAX**. Operating at Max torque overrides the Auto Reverse function.