


**QUICK GUIDE TO RESET
AEU-25T (ENDO DTC) TULSA/DENTSPLY**

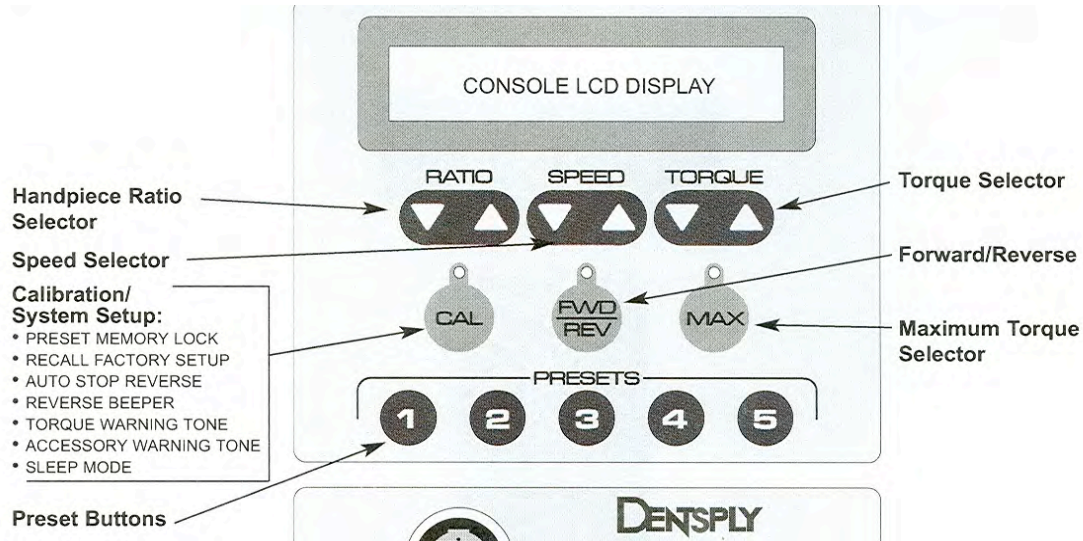

RESET BUTTON #5 FOR ENDO-EXPRESS (pages 7,8,13)

1. Turn on power - switch is on the back panel.
2. Turn **off** the “Preset memory lock.”
 - A. Press and hold the “CAL” button until you hear a double beep to enter the SetupMenu.
 1. PRESET MEMORY LOCK 1=ON, 2=OFF, 3=ESC, **PRESS #2.**
 2. RECALL FACTORY SETUP 1=NO, 2=YES, 3=ESC, **PRESS #1.**
 3. AUTO STOP REVERSE 1=ON 2=OFF, 3==ESC, **PRESS #3.**
 4. SAVE CHANGES, **SELECT 1.**
3. Reset button #5 for Endo-Express.
 - A. Set the Handpiece Ratio Selector to **1/4.**
 - B. Set the Speed to **6000.**
 - C. Set the Torque to **MAX.** Operating at MAX overrides the Auto Stop Reverse feature.
 - D. **PRESS AND HOLD BUTTON #5** until you hear the double beep.
 1. ACCESS FILE LIBRARY? 1=YES, 2=NO, 3=ESC, **PRESS #2.**
 2. RECORD PRESET? 1=YES, 2=NO, 3=ESC, **PRESS #1.**
 3. DISPLAY SHOULD SHOW:
USER DEFINED PRESET
1/4 AG 6000 MAX
4. Turn **on** the “Preset Memory Lock.”
 - A. Press and hold the “CAL” button until you hear the double beep to enter the Setup Menu.
 1. PRESET MEMORY LOCK 1=ON, 2=OFF, 3=ESC, **PRESS #1.**
 2. RECALL FACTORY SETUP 1=NO, 2=YES, 3=ESC, **PRESS#3**
 3. SAVE CHANGES, **SELECT 1.**