


QUICK GUIDE TO OPERATING THE ATR TECNICA DIGITAL TORQUE CONTROL MOTOR

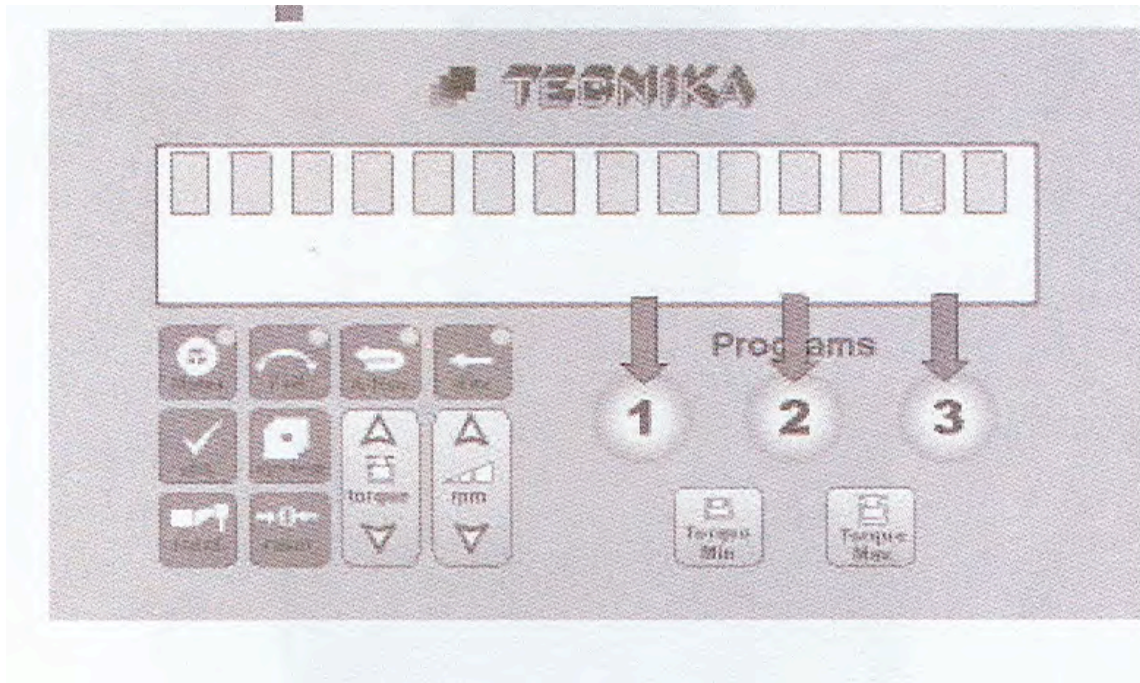

Setting the ATR Tecnika motor for ENDO-EXPRESS:

1. **Press button #3** (select GT Accessory Mode) and record the current settings on display.
2. **Press “redct”**, toggle through ratios to get to **1:1**.
3. **Press “rpm”** up/down button, raise rpm to maximum **12,800**.
4. **Press “Torque Max”** button (select Torque Max set mode).
5. **Press “torque”** up/down button, raise torque to maximum **100 ft/lbs**.
6. Turn off Auto Reverse by **toggleing the “A/Rev”** button to turn off the yellow light on the button.
7. **Press “reset”** button to lock in new settings – the motor is now ready to operate the ENDO-EXPRESS handpiece.