


QUICK GUIDE TO OPERATING THE ATR VISION TORQUE CONTROL MOTOR & SMART CARD

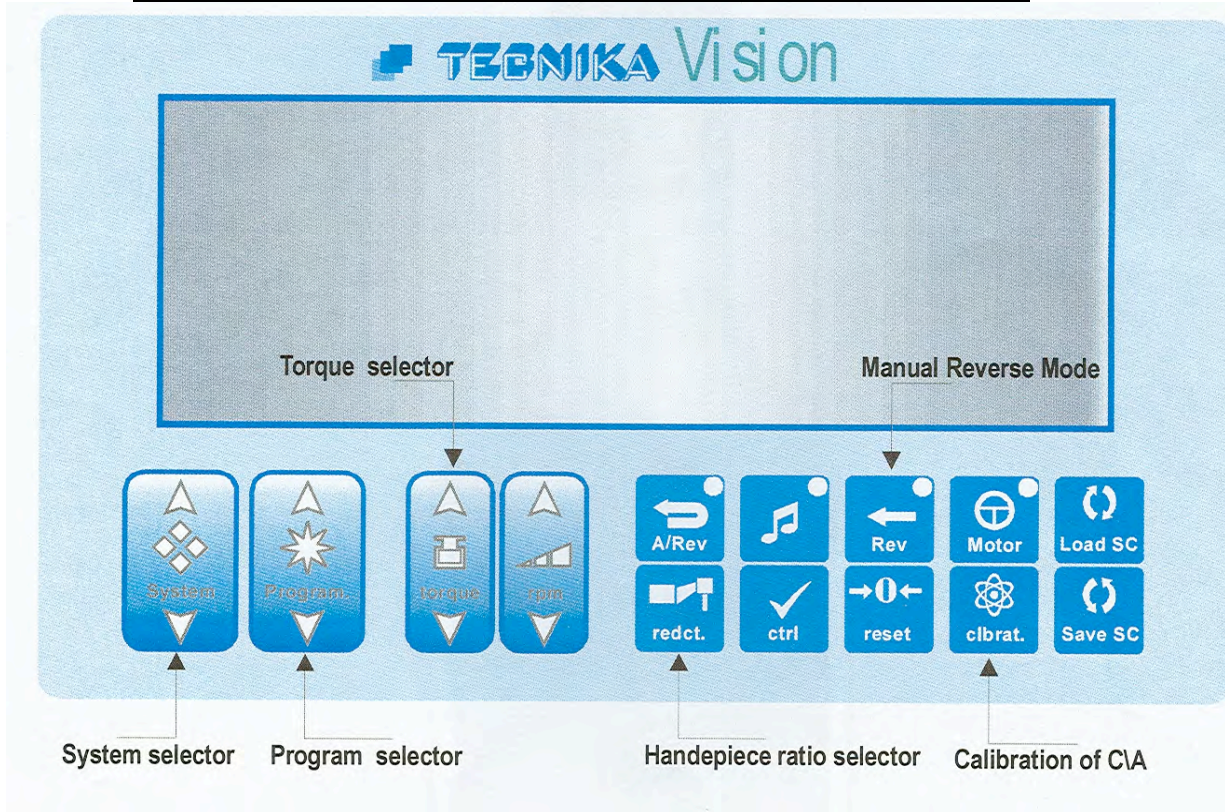

Setting the ATR Vision motor to use with ENDO-EXPRESS:

1. Press **“System”** and select GT Accessory mode and record the motors current settings on display.
2. Press **“Program”** and select **any** instrument.
3. Press **“torque”** up/down button to **maximum**.
4. Press **“rpm”** up/down button to maximum **12,800**.
5. Press **“redct”** button, toggle through ratios to get to **1:1**.
6. Turn off the Auto Reverse Function by **pressing “A/Rev”** button to turn off the yellow light on the button.
7. Press **“motor”** on/off button to use with the foot pedal.
8. Press **“reset”** button to lock in the new settings – the motor is now ready to use with ENDO-EXPRESS.
9. After the demonstration, you may reset the motor to its original settings – **press “ctrl + reset”** to remove temporary settings and return to initial settings.

SMART CARD (Data Card)

After #8 above:

- A. Insert the card and **press “save SC”** to save the ENDO-EXPRESS settings.
- B. Insert the card and **press “load SC”** to download the ENDO-EXPRESS settings inside the internal memory.