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Instructions For Use: **Tango-Endo™ Instruments**

89 Leuning Street, S. Hackensack, NJ 07606
www.edsdental.com • Made in USA

U.S. Patent No. 6,042,376 and other foreign patents pending.

Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.

Authorized European Representative:
Regulatory Affairs ONLY:
European Device Solutions
33, Stapleford Road,
Middlesbrough, Cleveland,
United Kingdom.
Contact E-Mail: TQMUK@aol.com

English

1. Indications for Use:

Tango-Endo™ instruments are for use in endodontic treatment for the removal of dentin and shaping of the root canal system.

2. Contra indications: As common with all mechanically driven root canal instruments, Tango-Endo™ instruments should not be used in cases of severe and sudden apical curvatures. In these cases EDS recommends the use of the SafeSiders® system.

3. Warnings:

Caution, Consult Instructions for Use **NON-STERILE**

- Instruments are packaged non-sterile, and must be sterilized before use.
- These instruments may contain Nickel Titanium alloy or Chromium.
- Do not use for individuals with known allergic sensitivity to these metals.
- The number of times the instruments can be re-used is dependent on treatment and procedure. It is recommended that you use these instruments no more than 3 times each before discarding. Instruments need to be sterilized and inspected before and after use.

4. Precautions:

As with all new products and procedures, you must exercise caution until you become proficient in its use. The reciprocating movement of the Tango-Endo™ handpiece helps to prevent binding. However, please note the following:

- A. The Tango-Endo™ instruments are only intended to be used in the Tango-Endo™ handpiece. They are never intended to be used in rotation.
- B. Operate the handpiece between 2500 - 3000 RPM (Revolutions per minute). The handpiece operates with a 4:1 reduction. Therefore, set your motor speed at 10,000 - 12,000 RPM.
- C. You must create a glide path prior to using Tango instruments. EDS recommends the use of the SafeSiders for glide path creation.
- D. Use a gentle inward pecking motion, with short 2 – 3 mm amplitude strokes. Always utilize minimal apical pressure. Do not force the instruments down the canal.
- E. Wipe the flutes clean frequently during instrumentation.
- F. Frequently irrigate and lubricate the canal throughout the procedure.
- G. Exercise caution in the apical area and around canal curvatures.

5. Adverse Reactions: None known.

Step-by-Step Instructions for Use:

6. Sterilization

Instruments must be sterilized before use.

- Scrub the instruments with soap and warm water
- Rinse thoroughly with distilled or deionized water
- Allow to air dry
- Place the instruments, unwrapped, in the autoclave tray
- Use fresh distilled or deionized water
- Steam sterilize per gravity procedures below:

- Temperature: 250° F/121° C
- Cycle Time: 20 minutes
- Maximum Dry Time: 10 minutes

OR

- Temperature: 270° F/132° C
- Cycle Time: 15 minutes
- Maximum Dry Time: 10 minutes

- Inspect instruments prior to use for any abnormalities.
- Recommended Instruments Disposal: Place used instruments in Biohazard Sharps container.

7. Instrumentation Sequence

- A. Prepare straight line access to canal orifice.
- B. Explore the canal using small-sized hand instruments or SafeSiders, determine working length, verify patency and confirm a smooth, reproducible glide path. EDS recommends the use of SafeSiders for glide path creation.
- C. Always irrigate and enlarge the glide path to a minimum 20/02 preparation.
- D. In the presence of irrigant, follow along the glide path and shape the wider extensions of oval canals with the Tango 30/02 shaping instrument (Single blue band on the latch head). Use a gentle inward pecking motion, with short 2 – 3 mm amplitude strokes until the working length is reached. Clean the flutes frequently during instrumentation and inspect for unwinding or abnormalities. Always utilize minimal apical pressure. Do not force the instrument down the canal.
- E. Use the Tango 30/04 finishing instrument (Double blue band on the latch head) in the same manner as described in step 7D. (Note: If you cannot achieve length, repeat step "D" above.)
- F. Confirm the Tango Finishing instrument reaches the apical measurement. Move the instrument continuously against the canal walls until there is no resistance. The precision Tango-Endo™ gutta percha point is now fit. (Note: Verify the gutta percha point fits to length).

8. Obturation

The Tango instrumentation system works with any obturation system (vertical, lateral condensation, master point, thermo-plastic). EDS recommends the use of EZ-Fill® or EZ-Fill® Xpress root canal cement in conjunction with Tango-Endo™ precision gutta percha points.

NOTE: For more information, videos, and FAQ's on this and all EDS products and techniques, please visit www.edsdental.com