

Endo-CHX[®]

2% Chlorhexidine Gluconate Solution
With Surfactant

Endo-CHX is intended to irrigate and cleanse root canal systems

Directions for Use:

- Use 17% EDTA as a pre-rinse in the canal to remove the smear layer.
- Rinse with Sodium Hypochlorite (NaOCl).
- Rinse out the NaOCl with sterile water or anesthetic solution.
- Place ENDO-CHX into an irrigating syringe.
- Continuously irrigate for a minimum of 2 minutes.
- Dry & Obturate the canal.

Warning: Not to be used as an oral rinse. Corrosive. Keep out of the reach of children. Avoid contact with eyes.

REF

Cat. No.

750-16

Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.
MADE IN USA

To download SDS (Safety Data Sheets)
Please Visit: <http://edsdental.com/sds>

16oz. (473mL)



89 Leuning Street, S. Hackensack, NJ 07606

1-800-22-FLEXI

TEL: (201) 487-9090
FAX: (201) 487-5120

E-MAIL: info@edsdental.com
www.edsdental.com



CHX16OZIN-2