

EDTA Plus is a root canal cleanser designed to facilitate removal of dental debris from the walls of root canals prior to obturation. EDTA Plus is a color enhanced, aqueous irrigating solution of 17% sodium EDTA with cetrimide surfactant.

Recommended use:

Shake before use. Irrigate the root canal during endodontic procedures using EDTA Plus in an irrigating syringe. Insert a side-vented needle approximately half way down the root canal. Thoroughly irrigate the canal and let stand for 60 seconds. Before obturation, rinse with sodium hypochlorite and dry the canal.