## The EZ-Change® Keeper & Cap Inserts For The Flexi-Overdenture® System

**ENGLISH:** EDS has developed the EZ-Change® Overdenture Attachment to facilitate easy replacement after wear. It consists of a metal keeper that is permanently cemented into the underside of the denture. The metal keeper has a space to receive a nylon cap insert that is threaded into place. When this nylon cap insert wears out, it takes just a few seconds to unscrew the worn out nylon cap insert from the metal keeper and replace it with a new one.

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### PLACEMENT TECHNIQUE

**1. ENGLISH:** The Flexi-Post Overdenture® posts are placed in the usual way. When the post fits properly within the post space, do not yet cement it in. It is important to make sure that when placed, that there is enough vertical space for the metal keeper and the nylon cap insert to be seated fully within the denture. To be sure, when the metal keeper with the nylon cap insert is first placed onto the ball attachment, prior to incorporation within the body of the denture, place marking spray onto the coronal surface of the metal keeper and place the denture over it. Remove the denture to view the area of impingement. Relieve the denture until there is no impingement and the denture is sitting passively on the ridge. In some cases it is not possible to relieve the denture from impingement without perforating the denture in an aesthetically crucial area. If this is the case, remove the uncemented post from the root and deepen the root-faced preparation. This allows for deeper seating of the post and more vertical clearance alleviating the possibility of an unaesthetic perforation of the denture.

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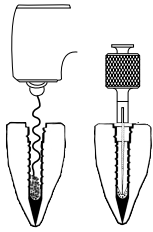
**2. ENGLISH:** Once adequate vertical clearance is established, cement the post with EDS' composite cement, Flexi-Flow® with titanium (Catalog # 850-00) or Flexi-Flow® Natural (Catalog # 860-00).

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**3. ENGLISH:** Place the metal keeper with the nylon cap insert onto the ball, making sure that the rubber band is in place.

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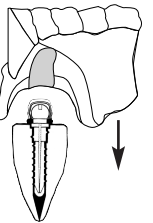
**4. ENGLISH:** Cold cure the assembled metal keeper (with the nylon cap insert) into the underside of the denture.

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
**5. ENGLISH:** Remove the denture and relieve the excess acrylic. You may now remove the rubber band from the post's head.

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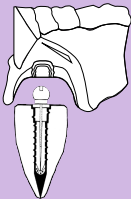continued on reverse side . . .

REPLACEMENT OF WORN  
NYLON CAP INSERT

6. **ENGLISH:** When the nylon cap insert wears out, take the two pronged wrench and place it into the two holes inside the nylon cap insert. Rotate the wrench counterclockwise for removal.

**IMPORTANT:** If the two holes are worn out from internal wear, heat the metal pronged wrench until the prongs are red hot and insert it anywhere into the nylon cap insert. The heated wrench will melt into the nylon creating its own holes. Allow a few seconds for the nylon to cool and solidify around the wrench. Then rotate the wrench counterclockwise until the nylon cap insert is removed. Upon first removal and final seat, a tell-tale tactile click will be evident for the dentist to know when placement is complete.

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**Flexi-Overdenture Kits and Their Contents**  
To order The Flexi-Overdenture System and accessories contact your authorized EDS dealer or call EDS.

**Flexi-Overdenture Stainless Steel Introductory Kits:**  
(12 Overdenture posts - 6 ea. of sizes 1 & 2, 12 nylon attachment caps, 12 rubber bands, 2 primary reamers, 2 countersink drills, 1 wrench, 1 depth gauge and 5 EDS gates glidden drills)\* ..... Cat. No. 210-00  
  
(4 Overdenture posts - 2 ea. of sizes 1 & 2, 4 nylon attachment caps, 4 rubber bands, 2 primary reamers, 2 countersink drills, 1 wrench, 1 depth gauge and 5 EDS gates glidden drills)\* ..... Cat. No. 211-00

**Flexi-Overdenture Titanium Introductory Kits:**  
(4 Overdenture posts - 2 ea. of sizes 1 & 2, 4 nylon attachment caps, 12 rubber bands, 2 primary reamers, 2 secondary drills, 1 wrench, 1 depth gauge, 4 brass transfer studs and 5 EDS gates glidden drills)\* ..... Cat. No. 212-00  
  
(12 Overdenture posts - 6 ea. of sizes 1 & 2, 12 nylon attachment caps, 12 rubber bands, 2 primary reamers, 2 secondary drills, 1 wrench, 1 depth gauge, 12 brass transfer studs and 5 EDS gates glidden drills)\* ..... Cat. No. 215-00

**(For Indirect/coping & Direct/non-coping Technique)**  
(4 Overdenture posts - 2 ea. of sizes 1 & 2, 4 nylon attachment caps, 4 rubber bands, 2 primary reamers, 2 secondary drills - for indirect/coping), 2 countersink drills - for direct/non-coping), 1 wrench, 1 depth gauge, 4 brass transfer studs and 5 EDS gates glidden drills)\* ..... Cat. No. 212-01  
  
(12 Overdenture posts - 6 ea. of sizes 1 & 2, 12 nylon attachment caps, 12 rubber bands, 2 primary reamers, 2 secondary drills - for indirect/coping) 2 countersink drills - for direct/non-coping), 1 wrench, 1 depth gauge, 12 brass transfer studs and 5 EDS gates glidden drills)\* ..... Cat. No. 215-01

**Flexi-Overdenture Stainless Steel Refills:**  
(6 OVD posts, 6 attach. caps, 6 rubber bands, 1 primary reamer and 1 countersink drill)\* Size 1 ..... Cat. No. 230-01  
Size 2 ..... Cat. No. 235-02  
Size 3 ..... Cat. No. 230-03

**Flexi-Overdenture Titanium Refills:**  
(6 OVD posts, 6 attach. caps, 6 rubber bands, 1 primary reamer, 1 secondary drill and 6 brass trans. studs)\* ..... Size 1 ..... Cat. No. 235-01  
Size 2 ..... Cat. No. 235-02  
Size 3 ..... Cat. No. 235-03

**(For Indirect/coping & Direct/non-coping Technique)**  
(6 OVD posts, 6 attach. caps, 6 rubber bands, 1 primary reamer and 1 countersink drill direct/non-coping)\* ..... Size 1 ..... Cat. No. 236-01  
Size 2 ..... Cat. No. 236-02  
Size 3 ..... Cat. No. 236-03

**Flexi-Overdenture Accessories:**  
(attachment caps 6 nylon female caps) ..... Cat. No. 290-01  
  
(OVD wrench) ..... Cat. No. 290-02  
  
(stainless steel depth gauge) ..... Cat. No. 290-06  
  
(titanium depth gauge) ..... Cat. No. 290-07  
  
(OVD countersink drill for second tier & flange preparation of titanium posts) ..... Size 1 ..... Cat. No. 292-01  
Size 2 ..... Cat. No. 292-02  
Size 3 ..... Cat. No. 292-03  
  
(6 transfer studs for titanium Overdenture posts)  
Size 1 ..... Cat. No. 293-01  
Size 2 ..... Cat. No. 293-02  
Size 3 ..... Cat. No. 293-03  
  
**Flexi-Overdenture EZ-Change Accessories:**  
(2 keepers, 2 cap inserts, 2 rubber bands & EZ-Change® wrench)\* ..... Cat. No. 250-00  
  
(2 keepers, 2 cap inserts & 2 rubber bands) \* ..... Cat. No. 250-01  
  
(6 nylon replacement cap inserts) ..... Cat. No. 250-02  
  
(EZ-Change® wrench) ..... Cat. No. 250-03

\* COMPLETE INSTRUCTIONS INCLUDED

For the latest product  
information, please visit  
[www.edsdental.com](http://www.edsdental.com)

ESSENTIAL  
DENTAL  
SYSTEMS

89 Leuning Street, S. Hackensack, NJ 07606,  
U.S.A.201-487-9090 • [info@edsdental.com](mailto:info@edsdental.com)

U.S. Patent No. 5,413,480. Other Foreign Patents Pending.

EZCINST-4

# **Important Sterilization Procedures:** **Endodontic Instruments, Posts, Drills and** **Wrenches are non-sterile.**

Prior to use, bag materials in an autoclave safe pouch and sterilize with a gravity displacement autoclave for 15 minutes at 132°C with a minimum drying time of 30 minutes. When reprocessing instruments, drills, wrenches, or taps, remove debris and dry before sterilization. Remove debris/soil using the EDS recommended cleaning protocol. Disinfect using a thermal disinfectant unit (washer-disinfectant) at 90°C for 1 minute.

**For complete instructions see website.**

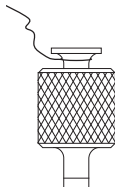
89 Leuning Street, S. Hackensack, NJ 07606  
Tel: 1-800-22-FLEXI, 201-487-9090 • Fax: 201-487-5120  
Web: [www.edsdental.com](http://www.edsdental.com) E-Mail: [info@edsdental.com](mailto:info@edsdental.com)

**Non-Sterile**

**(OVER)**

## Post Wrench Procedures:

In addition to the placement of a rubber dam, attach/tie a piece of dental floss (24 inch minimum) to the groove on the top of the wrench handle to allow for retrieval of the wrench.