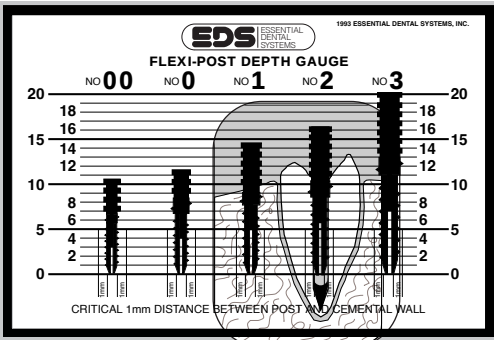
NOTE: If you are using Flexi-Post® for the first time, you need the wrenches, depth gauge and extender which are included in the Introductory Kit (Cat. No. 110-00 or 110-01), or may be purchased separately.

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1. ENGLISH: Use a depth gauge to determine optimum depth of the post. The second tier of the post must be fully seated to achieve maximum stability, retention and resistance to fracture.

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2. ENGLISH: Use a Peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal (EDS' Gates Glidden Drills are recommended). Then use the primary reamer.

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3. ENGLISH: Use the primary reamer to prepare the full length of the post hole (Cut Wet).

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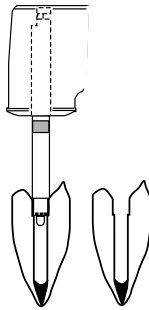
Caution: Federal law restricts this device to sale only to or by a licensed dentist.

STAINLESS STEEL POSTS
PROFFEN AUS ROSTFREIEM STAHL
PIVOTS EN ACIER INOXYDABLE
PERNI IN ACCIAIO INOSSIDABILE
ESPIGAS DE ACERO INOXIDABLE

Genuine Split-Shank System
Refill Instruction Sheet

ESSENTIAL DENTAL SYSTEMS


4. ENGLISH: Use the secondary drill to prepare the seat for the second tier. (The second tier must be fully seated. If not, you reduce the Flexi-Post's retention and increase the chance of fracture under function.)

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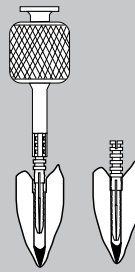
5. ENGLISH: Trial insert the post to determine fit.

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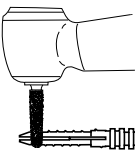
6. ENGLISH: Post adjustment: cut apical end, if necessary, to ensure seating of the second tier. (Cut both legs at the same time). Note: apical adjustment is always done after trial insertion.

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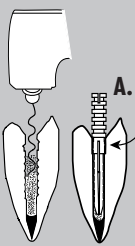7. ENGLISH: Post cementation. **Note fully seated second tier (A).** (Flexi-Flow Cem® from EDS is recommended). Follow with core formation and preparation (Ti-Core® composite Core build-up material from EDS is recommended).

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1-800-22-FLEXI
Tel: (201) 487-9090

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.
MADE IN USA

FPRSSIN-7

Important Sterilization Procedures: **Endodontic Instruments, Posts, Drills and** **Wrenches are non-sterile.**

Prior to use, bag materials in an autoclave safe pouch and sterilize with a gravity displacement autoclave for 15 minutes at 132°C with a minimum drying time of 30 minutes. When reprocessing instruments, drills, wrenches, or taps, remove debris and dry before sterilization. Remove debris/soil using the EDS recommended cleaning protocol. Disinfect using a thermal disinfectant unit (washer-disinfectant) at 90°C for 1 minute.

For complete instructions see website.

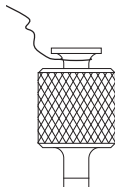
89 Leuning Street, S. Hackensack, NJ 07606
Tel: 1-800-22-FLEXI, 201-487-9090 • Fax: 201-487-5120
Web: www.edsdental.com E-Mail: info@edsdental.com

Non-Sterile

(OVER)

Post Wrench Procedures:

In addition to the placement of a rubber dam, attach/tie a piece of dental floss (24 inch minimum) to the groove on the top of the wrench handle to allow for retrieval of the wrench.